


# Multi-purpose Electric Operating Table



AOT8801A

## Hospital General Electric Foldable Removable Stainless Steel Operating Room Table

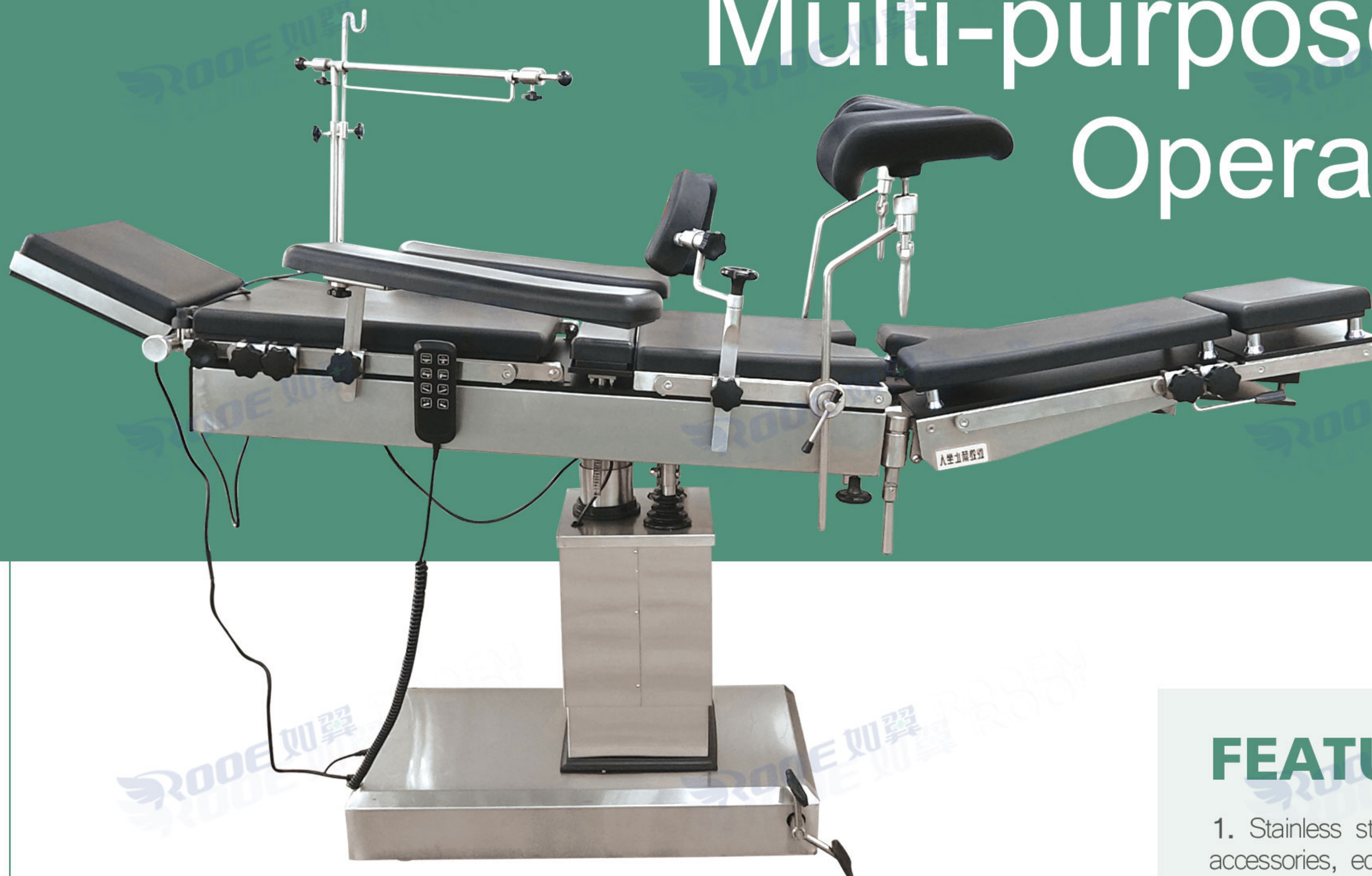

Ergonomic, can effectively reduce the labor intensity of medical staff. Applicable to general head, neck, chest, abdomen, perineum, limb surgery, as well as gynecology, ent, orthopedic surgery.

### FEATURES

1. Stainless steel side rails accept all kinds of available accessories, equipped with removable attachment clips, easy to use. Equipped with Anaesthetic screen, arm rest, body support, leg Support, foot plate. All fittings can jam the guide rail system.
2. The table is composed of head board, back bed board, waist bed board, sitting board, leg bed board and other basic accessories.
3. The back plate and leg bed plate adopt air spring design, which are detachable structure, and the head plate can be folded up and down; Leg bed plate can be folded down, outward, can be directly connected to the orthopedic traction frame.
4. The table is made of phenolic board, plexiglass and other materials to facilitate X-ray fluoroscopy of patients.
5. The removable memory sponge mattress, with unique support structure design, can be naturally shaped according to the patient's body temperature, effectively help patients disperse pressure, reduce the occurrence of bedsores, and relieve muscle fatigue and pain caused by long-term anesthesia.
6. The built-in waist bridge manual rocker lifts the waist, exposes the surgical field, and facilitates the adjustment of various positions required by lumbar spine, spine, kidney, gallbladder and other operations.
7. The base and column are covered with 304 stainless steel top to ensure the high durability of the operating room.
8. External 220V AC power supply, can be converted to 24V DC drive electric remote control by power controller. With internal battery, when the power supply production failure, can be temporary power supply from the internal power supply.



Tel: +86 512 58810663      Whatsapp/Mobile: + 86-18962287398; 15962399164  
Email: info@wanrooemed.com      Website: www.wanrooems.com  
Add: Room1310,1311, Guotai New World Plaza, No19, East Renmin Road, Zhangjiagang, Jiangsu, China.





# Functions Show

## Specifications

★ Various posture adjustments including table height adjustment, forward and backward, left and right tilt and vertical translation of the table are powered by electric remote control device.

★ Handheld electric remote control adopts 24V DC, easy to operate, safe and reliable.

★ Before and after the table electric translation  $\geq 400\text{mm}$ , can be equipped with c-arm, X-ray inspection.

★ The backplane adopts air spring structure, the backplane can be folded up to  $45^\circ$ , manually adjustable.

★ Four 75mm silent casters are used to move the operating table, equipped with brake foot, and the casters are locked or released by the foot lever.

★ Table bearing weight: 200kg




Model	AOT8801	AOT8801A	AOT8802
Backward and forward movements	$\geq 400\text{mm}$	$\geq 400\text{mm}$	None
Back section raised from the horizontal	None	$\geq 45^\circ$	$\geq 45^\circ$
Back section lowered from the horizontal	None	None	$\geq 15^\circ$



# Detail Description


## Operating table mattress

The table is made of phenolic board, plexiglass and other materials to facilitate X-ray fluoroscopy of patients.

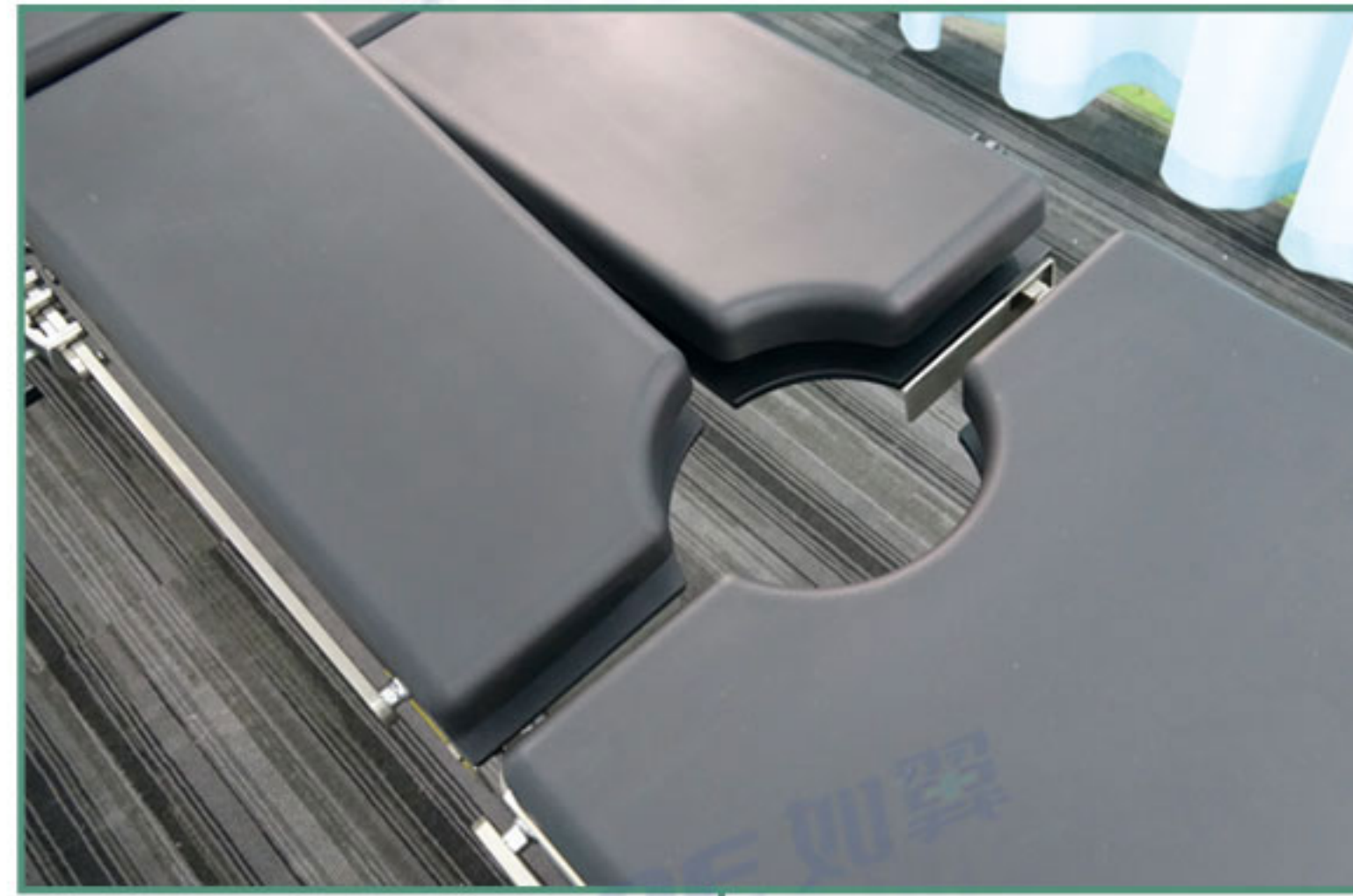

## Removable attachment clips

Stainless steel side rails accept all kinds of available accessories, equipped with removable attachment clips, easy to use.

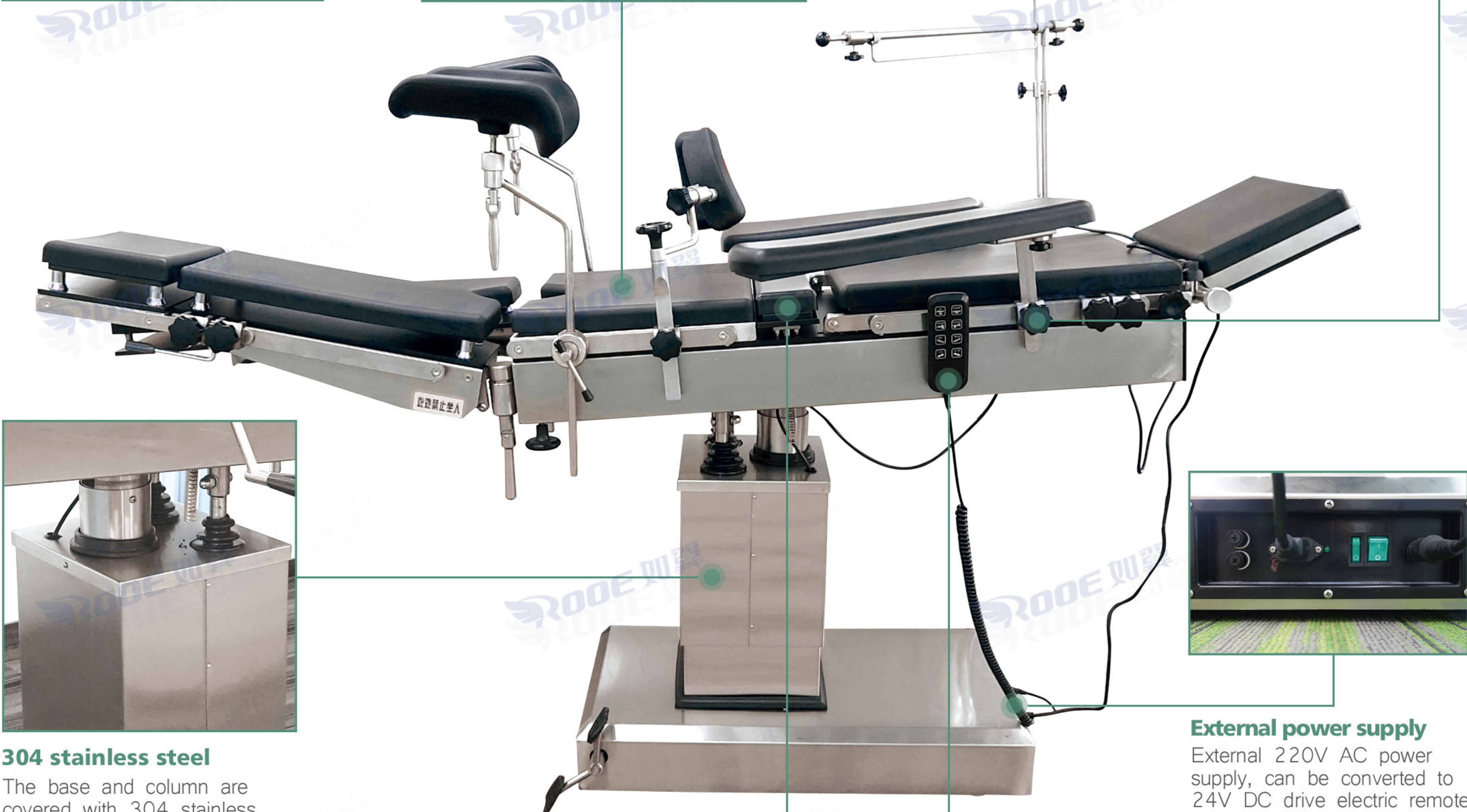

## 304 stainless steel

The base and column are covered with 304 stainless steel top to ensure the high durability of the operating room.



## External power supply

External 220V AC power supply, can be converted to 24V DC drive electric remote control by power controller. With internal battery, when the power supply production failure, can be temporary power supply from the internal power supply.



## Hand-held controller

Various posture adjustments including table height adjustment, forward and backward, left and right tilt and vertical translation of the table are powered by electric remote control device.

## Built-in waist bridge

The built-in waist bridge manual rocker lifts the waist, exposes the surgical field, and facilitates the adjustment of various positions required by lumbar spine, spine, kidney, gallbladder and other operations.



①



②



③



④



⑤


## Basic Accessories:

- ①. Anaesthetic screen: 1set
- ②. Arm support: 1pair
- ③. Body support: 1pair
- ④. leg support: 1pair
- ⑤. Cushion: 1set



# Postural adjustment

## AOT8801A

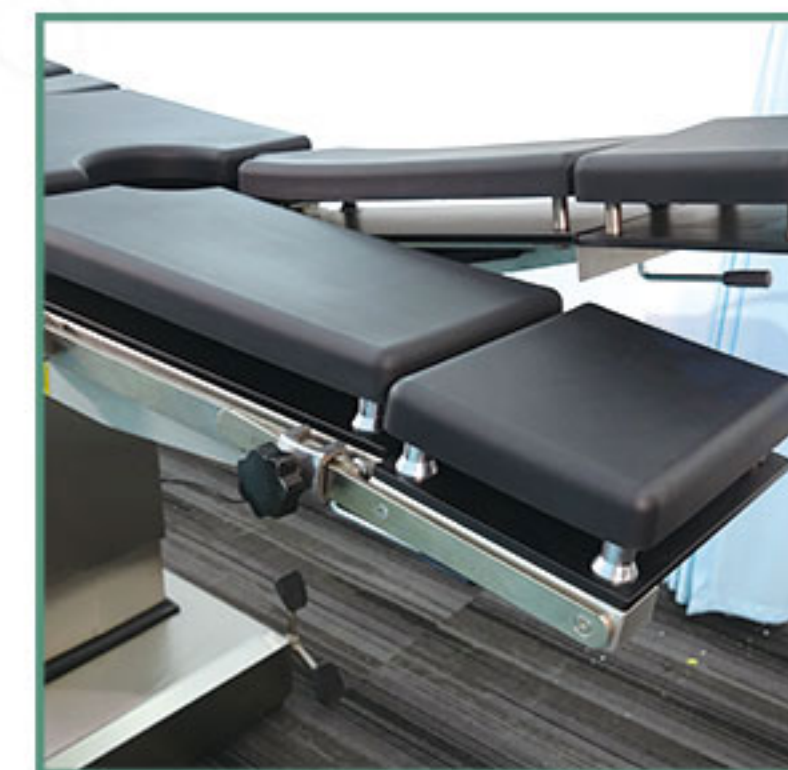


Applicable to general head, neck, chest, abdomen, perineum, limb surgery, as well as gynecology, ent, orthopedic surgery.



### General surgery

This operating table can satisfy the operation in the process of the human body each part examined the camera. Can cooperate C arm by Chest and abdomen, heart surgery, upper body X-ray inspection in the process of operation.

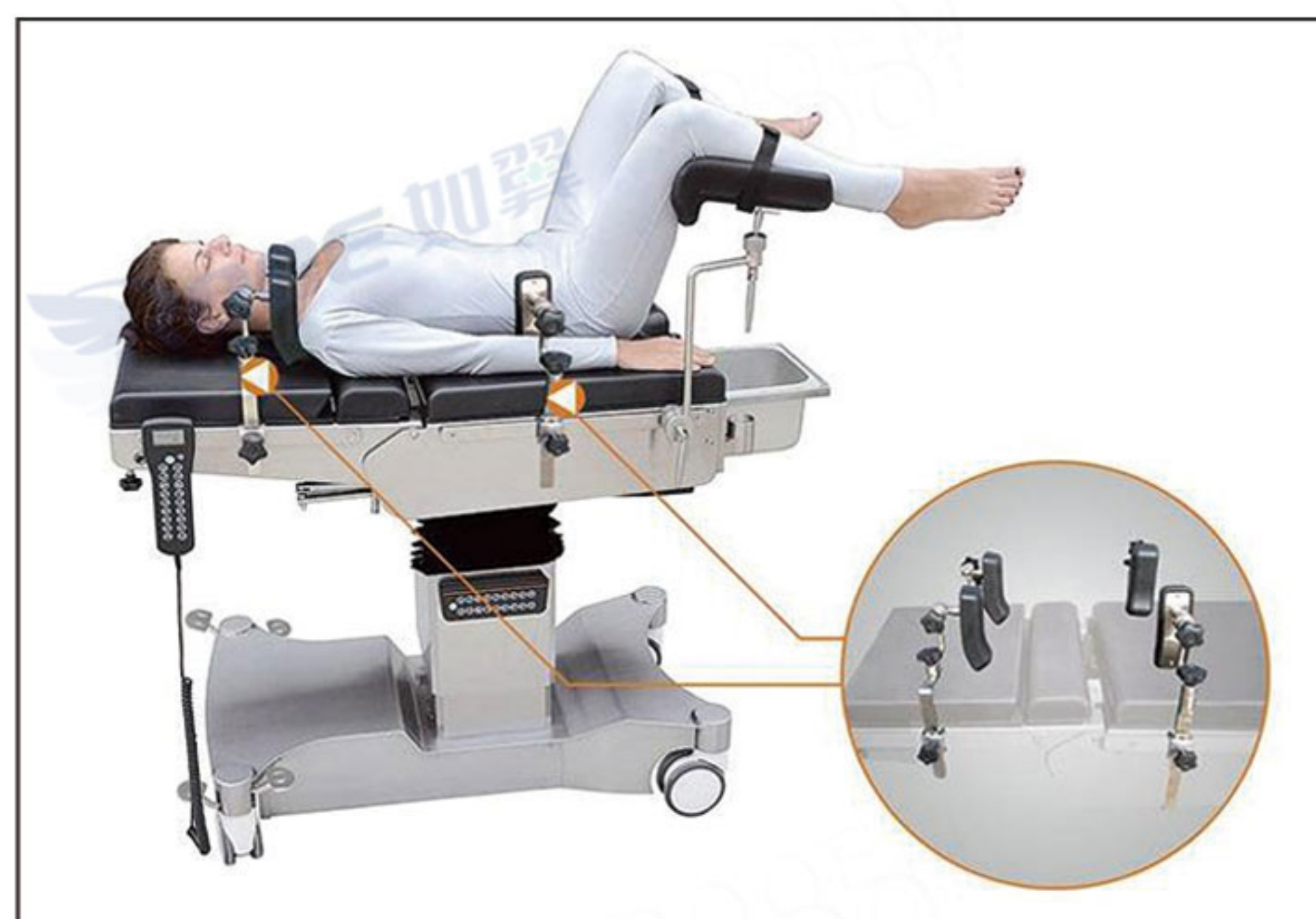

### Gynecologic/Obstetrics/urology surgery

The operating table leg with gas spring plate folded down. Fold down the leg plate 90° can be achieved without having to remove surgical purposes.



### Orthopaedic surgery

Orthopedic traction frame designed specifically for this operating table. Convenient and flexible, reliable locking. C-arm using a film with no blind spots.



### Thoracoabdominal surgery

The built-in waist bridge manual rocker lifts the waist, exposes the surgical field, raise the patient's chest surgery.



### Cardiovascular surgery

The longitudinal translation table 400mm, without moving the patient, surgery can be in the process of the whole body of ray inspection, Imaging of the human body without blind spots.



### otolaryngology / beauty supine surgery

The operating table can be reduced to a minimum 750mm, providing a comfortable height to doctors surgery. The unique chassis design of the operating table to the doctor to provide adequate legroom.



### Palmar traction operation



### Upper limb surgery



### Shoulder surgery



### Leg traction surgery



### Brain surgery (sitting)



### Knee surgery



# Postural adjustment

## AOT8801




It is suitable for chest, abdominal surgery, brain surgery, ophthalmology, otorhinolaryngology, obstetrics and gynecology, urology, orthopedics and other comprehensive operations in hospital operating rooms.



### General surgery

This operating table can satisfy the operation in the process of the human body each part examined the camera. Can cooperate C arm by Chest and abdomen, heart surgery, upper body X-ray inspection in the process of operation.

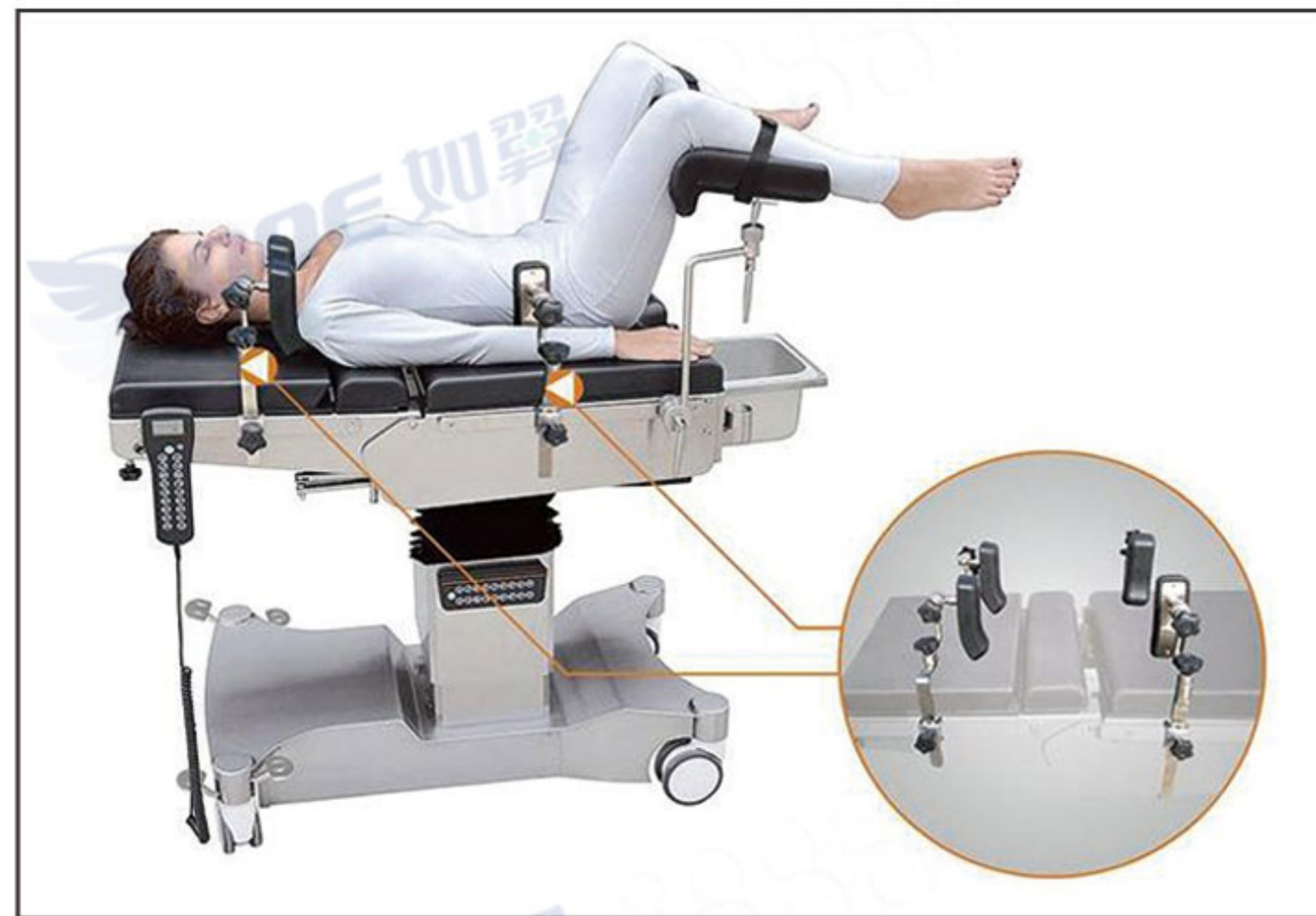

### Gynecologic/Obstetrics/urology surgery

The operating table leg with gas spring plate folded down. Fold down the leg plate 90° can be achieved without having to remove surgical purposes.



### Orthopaedic surgery

Orthopedic traction frame designed specifically for this operating table. Convenient and flexible, reliable locking. C-arm using a film with no blind spots.



### Thoracoabdominal surgery

The built-in waist bridge manual rocker lifts the waist, exposes the surgical field, raise the patient's chest surgery.



### Cardiovascular surgery

The longitudinal translation table 400mm, without moving the patient, surgery can be in the process of the whole body of ray inspection, Imaging of the human body without blind spots.



### Otolaryngology / beauty supine surgery

The operating table can be reduced to a minimum 750mm, providing a comfortable height to doctors surgery. The unique chassis design of the operating table to the doctor to provide adequate legroom.



### Palmar traction operation



### Knee surgery



### Leg traction surgery



### Upper limb surgery

Electric operating table is designed for all kinds of operations in hospitals. Beside all functions of the Chinese-build operating tables, it has its own advantages. Leg section can be rotated, opened, disassembled and adjusted easily. It is most suitable for the urological operation. The table top, made of high strength plastic plate, can move backwards and forwards. X-ray examination can be carried out on it. The table's elevation, lowering, reversed Trendelenburg; Trendelenburg, lateral tilt, and movement of table-top are all automatically by the button.



# Postural adjustment

## AOT8802




It is used for multi-position support and operation of patients in conventional surgery, surgery (neurosurgery, thoracic surgery, general surgery, urology), ent (ophthalmology, etc.), orthopedics, gynecology surgery and other medical procedures, making them lie down in different postures.



### General surgery

This operating table can satisfy the operation in the process of the human body each part examined the camera. Can cooperate C arm by Chest and abdomen, heart surgery, upper body X-ray inspection in the process of operation.

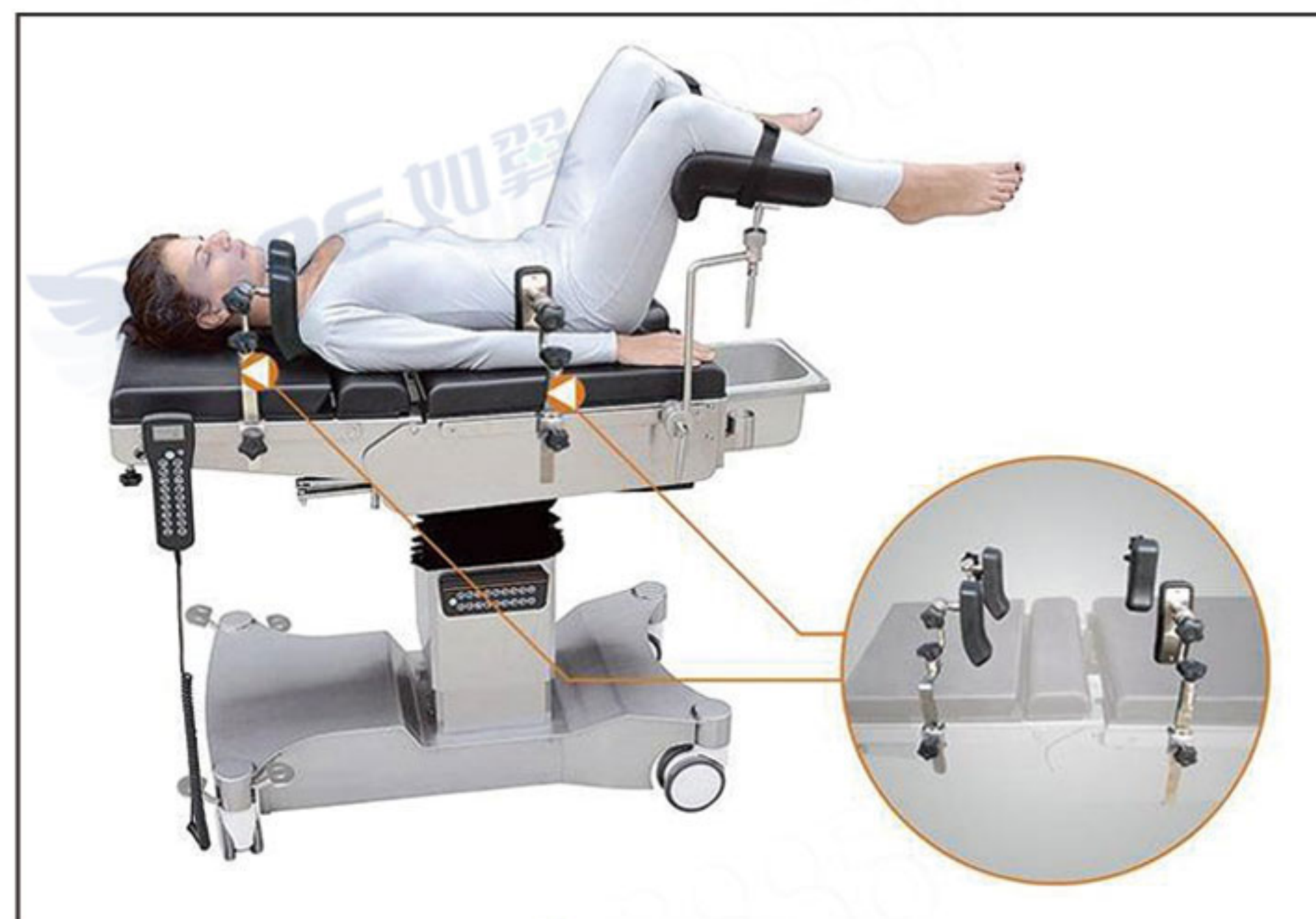

### Gynecologic/Obstetrics/urology surgery

The operating table leg with gas spring plate folded down. Fold down the leg plate 90° can be achieved without having to remove surgical purposes.



### Orthopaedic surgery

Orthopedic traction frame designed specifically for this operating table. Convenient and flexible, reliable locking. C-arm using a film with no blind spots.



### Thoracoabdominal surgery

The built-in waist bridge manual rocker lifts the waist, exposes the surgical field, raise the patient's chest surgery.



### Kidney surgery

After the patient side, must support the patient with a side support, by using the bed back and hip, bending back waist position to the patients, up to 12cm, mesa 220° bend is better for renal resection, built-in waist bridge can lift up when necessary.



### Otolaryngology / beauty supine surgery


The operating table can be reduced to a minimum 750mm, providing a comfortable height to doctors surgery. The unique chassis design of the operating table to the doctor to provide adequate legroom.



### Palmar traction operation



### Shoulder surgery

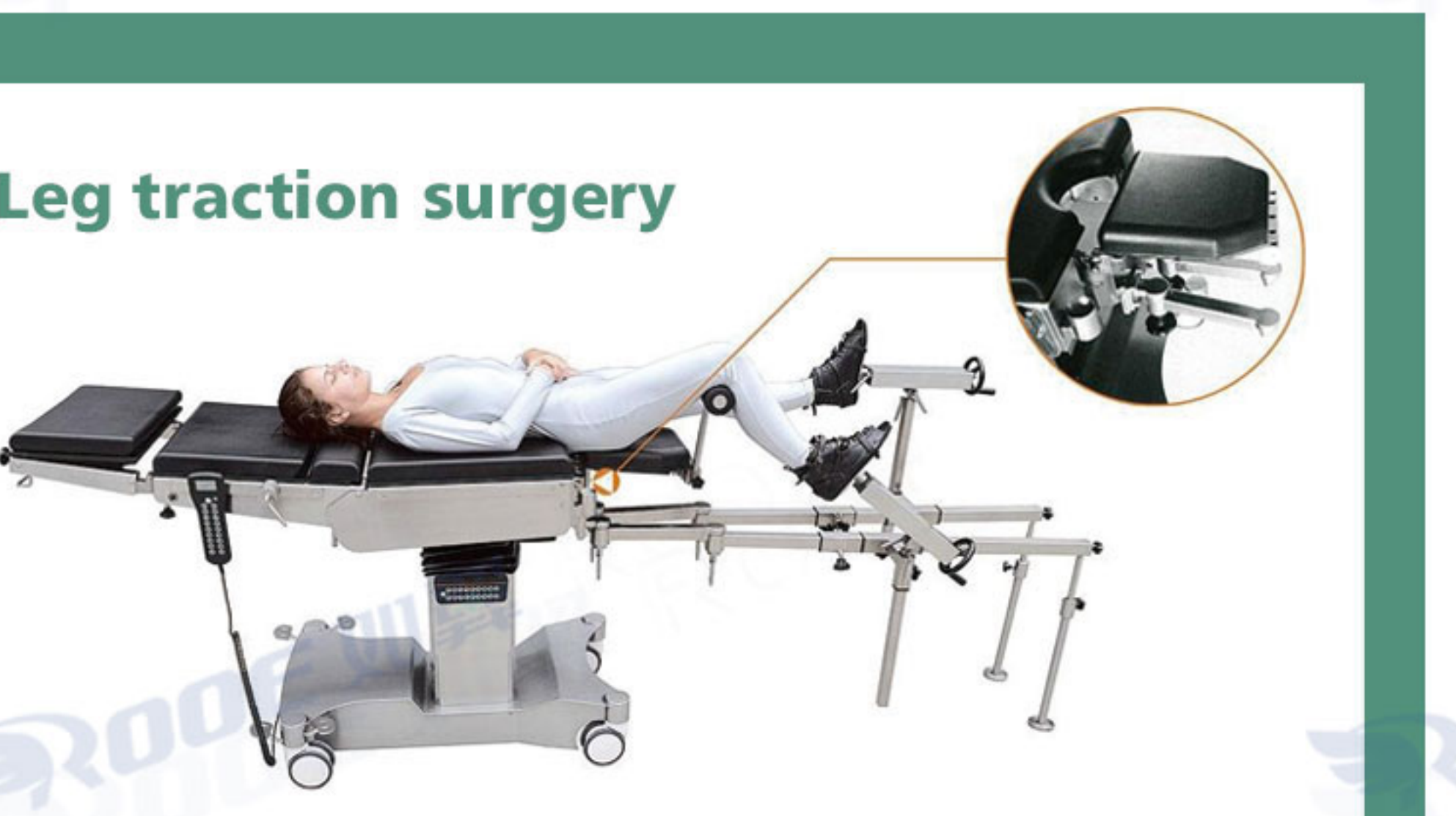

### Leg traction surgery



### Upper limb surgery



### Brain surgery (prone position)



### Brain surgery (sitting)




### Knee surgery



# Multi-purpose Electric OPERATING TABLE



AOT8801

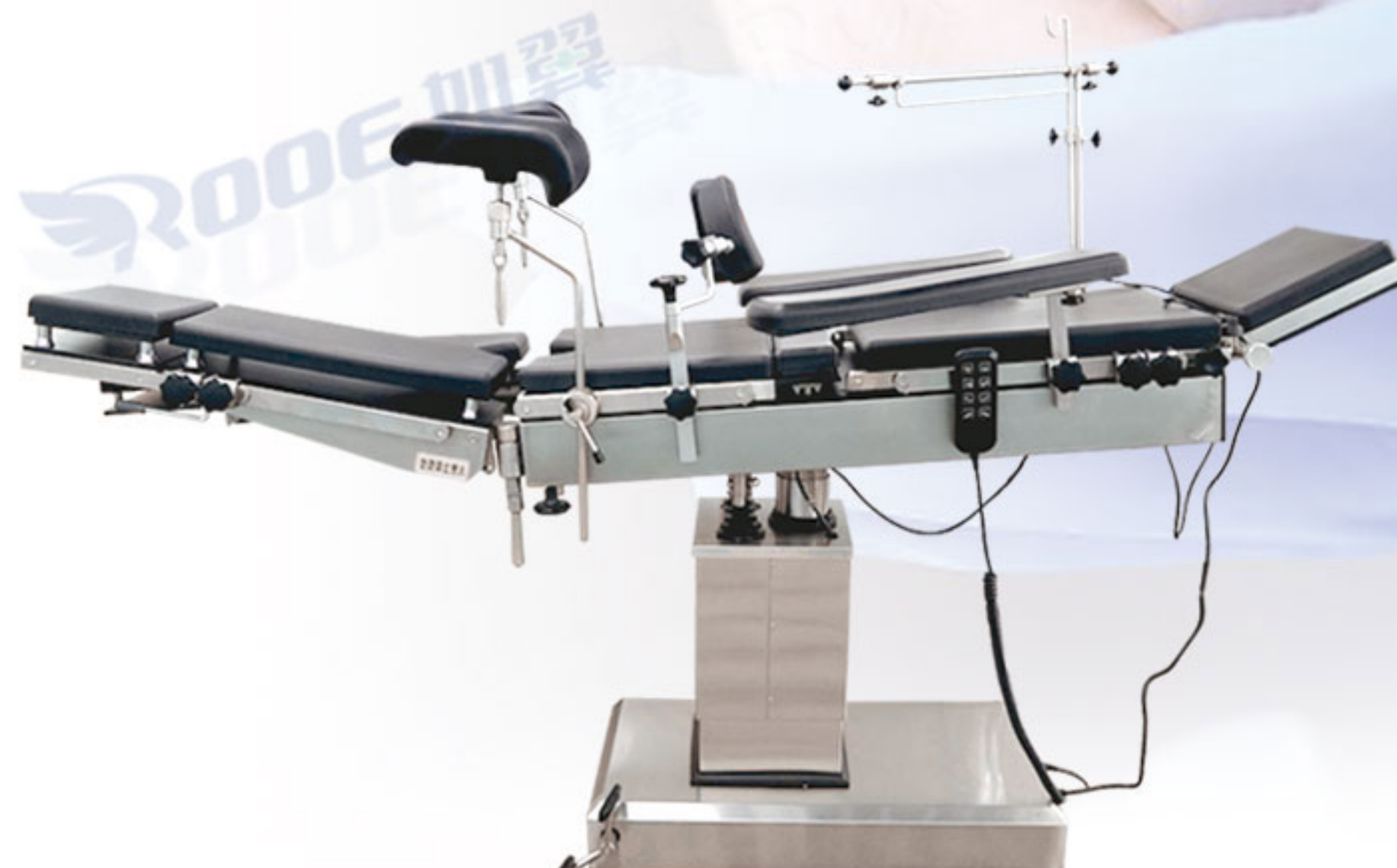

AOT8801A



AOT8802

Model		AOT8801	AOT8801A	AOT8802
Operating Table Size	Length x Width	2100x480mm	2100x480mm	2100±100x480±40mm
Sectional Dimensions Of Operating Table	Head Rest	280×300mm	280×300mm	280×300 (±20) mm
	Back board	500×480mm	500×480mm	500×480 (±20) mm
	Butt plate	450×480mm	450×480mm	450×480mm (±20) mm
	Leg board	500×230mm	500×230mm	500×230mm (±20) mm
	Foot board	200×260mm	200×260mm	200×260mm (±20) mm
Height Adjustable Range	Waist board	100×480mm	100×480mm	100×480mm (±20) mm
	Min. height of table top from floor	750mm	750mm	750mm
Height Adjustable Range	Max. height of table top from floor	1000mm	1000mm	1000mm
	Reversed tendelenburg	≥25°	≥25°	≥25°
Table Overall Adjustable Range	Tendelenburg	≥25°	≥25°	≥25°
	Lateral tilt	≥15°	≥15°	≥15°
	Backward and forward movements	≥400mm	≥400mm	None
Table Section Adjustment Range	Head section raised from the horizontal	≥45°	≥45°	≥45°
	Head section lowered from the horizontal	≥90°	≥90°	≥90°
	Kidney bridge elevated	≥120mm	≥120mm	≥120mm
	Back section raised from the horizontal	None	≥45°	≥45°
	Back section lowered from the horizontal	None	None	≥15°
	Leg section lowered from the horizontal	≥90°	≥90°	≥90°
Leg section opened	≥90°	≥90°	≥90°	
Power Supply		AC 220V±10%,50Hz		

★Foot board can be horizontal or vertical with leg board.

JIANGSU ROOE MEDICAL TECHNOLOGY CO.,LTD.

Tel:+86 512 58810663

Whatsapp/Mobile:+ 86-18962287398;15962399164

Email:info@wanrooemed.com Website: www.wanrooems.com

Add: Room1310,1311, Guotai New World Plaza, No19,East Renmin Road,Zhangjiagang, Jiangsu, China.



we supply customer one-stop purchasing solutions timely.